

## Door Control Station

Cat. 6103

### DESCRIPTION

Jeron's Cat. 6103 Door Control Station is used in the Pro-Alert™ 610 Nurse Call System to monitor door contacts.

- LED Alarm Indicator
- Key Removes in Both Positions

### OPERATION

When the keyswitch is turned to the ON position the door circuits are armed. If a door circuit changes state momentarily, the station's call placed indicator and Corridor Dome Light illuminate steady white. At the 6165 Nurse Master Station, the ROUTINE call placed indicator and the LED corresponding to the calling station will illuminate accompanied by a routine alert tone. The 6165 Nurse Master LCD will display the calling room number and the word DOOR ALARM.

If so programmed, the 6810 Zone and 6817 Duty Stations will activate.

To cancel a call, turn the keyswitch to OFF at the originating station

### EQUIPMENT

The 6103 Door Control is equipped with a green call placed indicator, and tumbler lock keyswitch with two keys.

The Door Control is furnished on a white, flame-retardant



modular single-gang subplate for flush wall mount with a separate thermoset trimplate. The panels resist common hospital cleaning agents.

### TECHNICAL DATA

#### Mounting:

- White thermoset plastic 1-gang trimplate with separate modular sub-plate and matching plastic hardware

#### Dimensions (H x W x D):

- 4.5 x 2.75 x 2.5 in. (11.4 x 7 x 6.4 cm)

#### Back Box Required:

- Raco 670RAC 1-gang or UL recognized equivalent

#### Controls/Indicators:

- Call placed indicator: Red LED
- Keyswitch with ON/OFF positions
- Keys remove in both positions and are keyed alike; two supplied

#### Operating Voltage:

- 28 VDC

#### Field Connections:

- Plug-in IDC block; Connector provided

#### Certification:

- UL 1069 Listed
- C-UL—Canadian Standard C22.2 NO.125

PRODUCT IS SUBJECT TO CHANGE WITHOUT NOTIFICATION.