



Health Care Call System

Staff Emergency Station Cat. 6131

DESCRIPTION

Jeron's Cat. 6131 Staff Emergency Stations are used in the Pro-Alert™ 610 System to signal staff for immediate assistance.

OPERATION

Pressing the top of the button activates all system call signals for an urgent-level call. The yellow call placed indicator illuminates, and the corridor Dome Light's yellow and green bulbs flash alternately. The Cat. 6165 Nurse Master LCD displays the room number and the preprogrammed call message. The yellow URGENT call placed indicator, accompanied by the urgent alert tone, and LED corresponding to the calling station flash at the Cat. 6165 Nurse Master Station.

If so equipped, the Cat. 6810 Zone Light and the Cat. 6817 Duty Stations will activate.

Pressing the bottom of the button at the originating station cancels the call.



EQUIPMENT

The station is furnished with a call button, yellow call placed LED, and 1/8" high 'STAFF EMERGENCY' label on a modular green plastic subplate for flush wall mount with included white plastic trimplate. The low profile pushbutton has a positive detent rocker action to resist accidental or partial operation. The panels resist common hospital cleaning agents.

TECHNICAL DATA

Mounting:

- White thermoset plastic 1-gang trimplate with separate green modular subplate and matching plastic hardware

Dimensions (H x W x D):

- 4.5 x 2.75 x 2.5 in.
(11.4 x 7 x 6.4 cm)

Back-box Required:

- Raco 670RAC 1-gang or equivalent mounted 1/2" behind wall face or Raco 232 2-gang with Raco 772 1-gang raised cover or equivalent

Controls/Indicators:

- Pushbutton: Black molded thermoplastic
- Call placed LED indicator: Yellow LED

Operating Voltage:

- 28 VDC

Field Connections:

- Plug-in IDC block; Connector provided

Certification:

- UL 1069 Listed
- C-UL — Canadian Standard C22.2 NO. 125

PRODUCT IS SUBJECT TO CHANGE WITHOUT NOTIFICATION